ProSomnus® Sleep and Snore Devices

INSTRUCTIONS FOR USE

ProSomnus EVO®
ProSomnus EVO [PH]
ProSomnus [IA]
ProSomnus [CA]
ProSomnus [PH]
ProSomnus MicroO₂®

U.S. Patent Nos. 9,808,327|9,820,882|9,949,868|10,213,280|10,603,207

<https://prosomnus.com/instructions/PRO3-267-G> (January 2023)

ProSomnus Sleep Technologies
5675 Gibraltar Drive
Pleasanton, CA 94588, USA
Phone +1 844 537 5337
ProSomnus.com

Scan for User Manual

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ENGLISH

Product Information

This is your prescribed ProSomnus Sleep and Snore Device ordered by your Dentist or Physician. Your Dentist or Physician will review the proper techniques and instructions for adjusting, maintenance and cleaning your sleep device. A supplemental User Manual is available at <https://prosomnus.com/instructions/>.

Indications

The ProSomnus Sleep and Snore Devices are intended to reduce night time snoring and mild to moderate Obstructive Sleep Apnea (OSA) in adults. Optionally, if the DentiTrac micro-recorder is completely embedded in the ProSomnus Sleep and Snore Device, the micro-recorder is intended to measure patient compliance to oral device/appliance therapy in combination with the DentiTrac System.

Contraindications

The device is contraindicated for patients who:

- Have Central Sleep Apnea
- Have severe respiratory disorders
- Have loose teeth or advance periodontal disease
- Are under 18 years of age

There are known hazards with using an oral sleep apnea device.

Have your Dentist or Physician review them with you.

Warnings

Use of the device may cause:

- Tooth movement or changes in dental occlusion
- Gingival or dental soreness
- Pain or soreness to the temporomandibular joints
- Obstruction of oral breathing
- Excessive salivation

Precautions

Patients should consider their medical history, including history of asthma, breathing, or respiratory disorders, or other relevant health problems, and contact the appropriate healthcare provider before using the device.

Storage

Keep out of reach of small children and pets.

The ProSomnus Sleep and Snore Device should be stored in a cool, dry place.

Ensure the device is not exposed to extreme temperature in excess of 60°C/140°F.

The printed ProSomnus Sleep and Snore Device Instructions for Use may be requested at:

+1 844 537 5337 or email Info@ProSomnus.com

Caution: Federal (U.S.) law restricts this device to sale by, or on the order of a Dentist or Physician.

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