

NEW!



INTRODUCING

ProSomnus® EVO® Guided

The first bioengineered oral device to dilate the velopharynx and the oropharynx.

4x lower profile
titration mechanism



Lowest 2.5mm vertical requirement
promotes mouth closure

95° guided post stabilizes mandible,
optimizes comfort, and encourages
mouth closure

The Solution

EVO Guided is bioengineered to prevent airway collapse by repositioning the hyo-glossal complex:

- Guide, stabilize and titrate the mandible into your prescribed therapeutic position.
- Guide the tongue to a beneficial anterior position.
- Guide the mouth closed.

Precision Sleep Medicine

100% made from the provider's oral records for each patient, with no prefabricated components, and no modifications from your prescription.

True anatomical splint design



Lingualless design offers up to 14x more
tongue space than predicate devices

Why EVO Guided?

Aim for 100% clinical success. Prescribe EVO Guided as your frontline device to optimize: Efficacy, Patient Satisfaction, Referrals, and Adherence. Mitigate Side Effects. Boost practice economics by reducing costly non-responder patients.

MG6™ Technology | ISO 10993 compliant | US Military drop test compliant | Less porous, biocompatible and hygienic | BPA Free | Proprietary engineering and manufacturing information.



REQUEST YOUR FREE STARTER KIT
AND LET'S CONNECT!

Advancement Guide

| 4 DEVICES U0,U2,L0, L1 | COMBINATION | ADVANCEMENTS |
|---|----------------------|--------------|
|  | Upper 0 + Lower 0 | 0mm |
|  | Upper 0 + Lower 1 | 1.0mm |
|  | Upper 2 + Lower 0 | 2.0mm |
|  | Upper 2 + Lower 1 | 3.0mm |

When ordering additional advancement arches, please use the following guide for the next recommended arch sequence:

U4 + L0 = 4.0mm

U4 + L1 = 5.0mm

U6 + L0 = 6.0mm

U6 + L1 = 7.0mm

U8 + L0 = 8.0mm

U8 + L1 = 9.0mm

